


Detect[®]2



Instrumenttien puhdistuksen patentoitu laadunvarmistusväline.

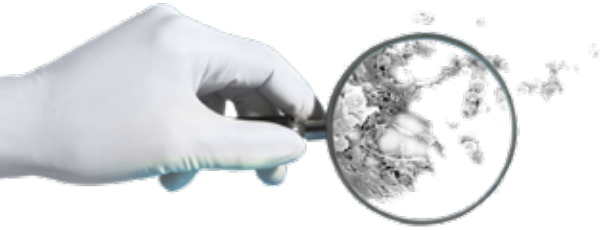

- Detect[®]2 on väline jolla varmistetaan leikkaus-instrumenttien puhdistuksen laatu
- Patentoitu ja validoitu teknologia biofilmin ja lian havainnointiin
- Tulokset nopeasti - nähtävissä jo 5 minuutissa suoraan instrumenttien pinnalta

Käyttökohteet

Ensisijaisesti ruostumattomasta teräksestä valmistetuille hammas- ja leikkausinstrumenteille




Tuoteominaisuudet

- Nopea ja yksinkertainen käyttää: korillinen instrumentteja vain 5 minuuttia (DIN 1/1)
- @oseleekpqorjkqqekm̄r̄kf ðekaelekkckhrk
- High sensitivity (from 10 µg/cm² protein) A single reagent for detecting residual proteins (dye process)
- Compatible with stainless steel, PP, PTFE, POM, aluminium, titanium
- Registered Medical Device Class I
- Biodegradability ≥ 90 % (OCDE 302B)
- Very low toxicity (reposts available on demand)
- Hajuton



Instructions for use

1. Immersion of instruments: 5 minutes in the COLORANT solution
2. Rinse thoroughly with tap water
3. Visual interpretation of results
Blue stains are indicative of residual proteins/biofilms

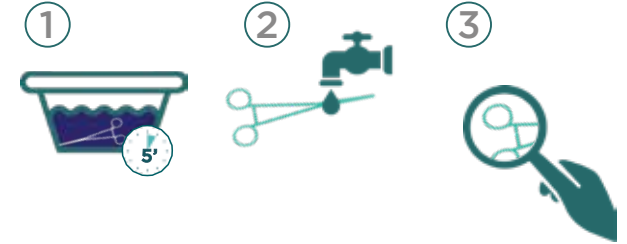

Muut ominaisuudet

Appearance: dark blue liquid
Density: 1,013 +/- 0,01
pH: 2,25 +/- 0,25

Pääainesosat

Citric acid: 1-5%
COLORANT/Dye
0,1-1% Alcohol free

Varotoimenpiteet

- **Wear gloves and a protective clothing against the dye.** Refer to the safety data sheet before use
- **Protect** the working area
- **Store** in the original container, closed, between +4°C and +25°C. For optimal performance respect the expiry date on the label
- **End of life:** an agreed collector must eliminate unused product

Toimituskoot

Ref OL22215
KIT
Ref OL22329
6 x 950ml
Ref OL22319
2 x 5L



Maahantuonti & markkinointi:

SYMPPIIS International Oy
PL 67, 02771 Espoo
puh. 020 735 3333
www.symppis.com - info@symppis.com

SYMPPIIS - Terveenä kotiin™

onelifebiofilmfree.com

